




HOW TO USE YOUR QUINGAS REGULATOR

1. Screw on the regulator to the top threads of the calibration cylinder.
2. Insert the syringe with the stopcock in the open position into the output of the regulator.
3. With the stopcock open, press the button on the top of the regulator to fill the syringe to at least 20ml.
4. Close the stopcock then remove the syringe and stopcock from the regulator.



Regulator

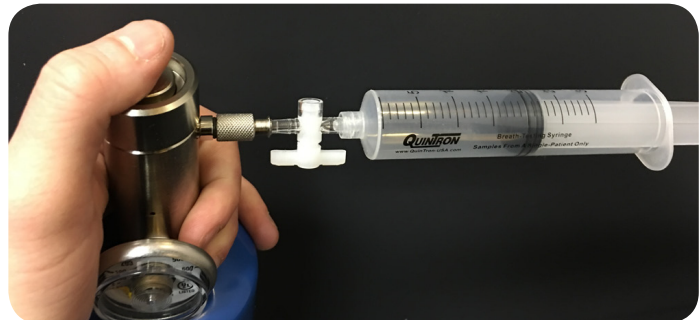

DISPOSAL INSTRUCTIONS

The valve removal tool is used to release the remaining gas in the QuinGas cylinder for safe disposal.

1. Remove the QuinGas regulator from the top of the cylinder.
2. Insert the valve removal tool into the small pressure release hole on the top of the cylinder next to the main threads.
3. Gently rotate the tool until it catches the valve stem.
4. Push down and twist in a counter-clockwise direction until the valve is completely removed.
5. The cylinder can now be disposed in accordance with your local regulations.

Do not dispose of the regulator or valve removal tool.

The cylinder is made of steel. Due to varying local regulations regarding recycling, refer to your community regulations or contact your local waste management company for more specific instructions regarding disposal.

Valve Removal Tool

